




CapKeeper® Test Strip Cap & Container Organizer Instructions For Use

K100-1250

fits 1-1/4" caps

K100-1375 — fits 1-3/8" caps

K100-1750 — fits 1-3/4" caps

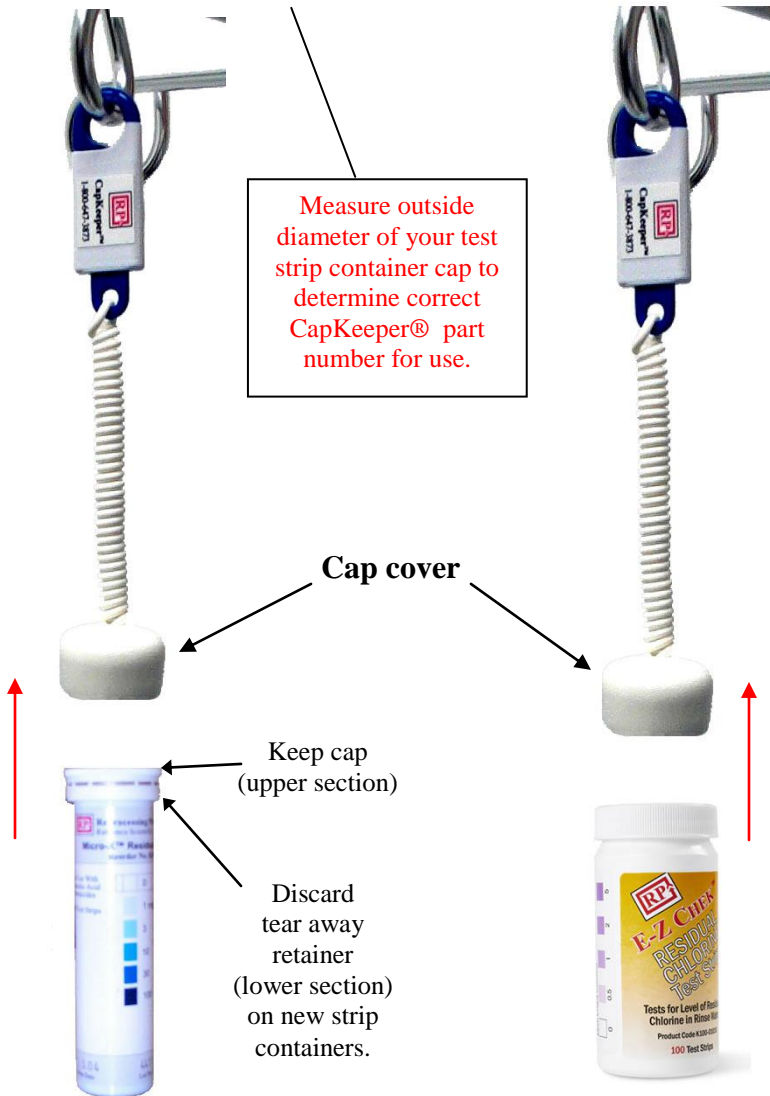

Push/pull-off Cap Style

1. Push capped test strip container completely into Capkeeper® cap cover.
2. To open/close strip container: grip and squeeze cap cover while pulling/ pushing on test strip container.

Rotate-on/off Cap Style

1. Push capped test strip container completely into CapKeeper® cap cover.
2. To open/close strip container: grip and squeeze cap cover while rotating test strip container.