



For Free Technical Support  
Information, Visit RPC's Web-site:  
[www.rpc-rabrenco.com](http://www.rpc-rabrenco.com)

## **CathGuard™**

### **Patient Catheter Protective Pocket**

### **Instructions for Use - P/N 7001-3920**

#### **Indications for Use:**


The RPC CathGuard™ Patient Catheter Protective Pocket is intended as a protective device for external central lumen catheters, and to provide comfort for the dialysis patient between dialysis treatments. Exterior catheter immobilization provided by the CathGuard™ may reduce trauma due to pulling at the central line site.

#### **Warnings and Precautions:**

CathGuard™ is a disposable device and regular replacement at each dialysis treatment is recommended. CathGuard™ is not intended for use as a central line site *dressing* and provides no protection to the site itself. CathGuard™ is non-sterile. Follow the instructions provided by your medical provider for central line dressing care. CathGuard™ is water-resistant but not water-proof.

#### **Instructions for use (see figure below):**

1. Carefully place the CathGuard™ pocket over the exterior lumen of the central line catheter.
2. Slide the catheter lumen and caps into the CathGuard™. Avoid pulling on the catheter site.
3. With the catheter in place inside the CathGuard™, remove the two adhesive strip covers on the back of the CathGuard™ and secure it to the patient's skin. Use of a skin prep will improve CathGuard™ adherence and may increase patient comfort by reducing skin irritation.

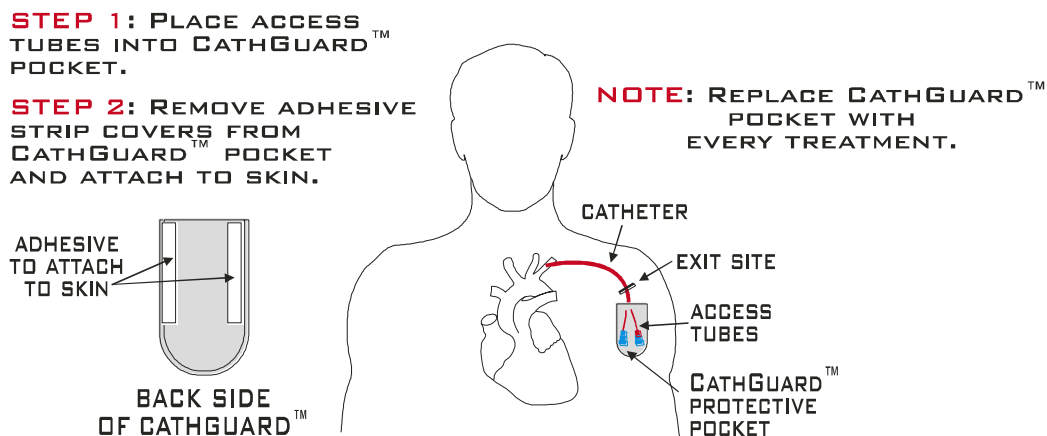

RPC • Tucson, AZ • Minneapolis, MN • USA • (520) 888-5551 • Fax (520) 888-5557

**Product Quality at Discount Pricing<sup>4</sup>**



012007A