



CathGuard™

Patient Catheter Protective Pocket

REF 7001-3920



Instructions for Use

Indications for Use:

CathGuard™ Patient Catheter Protective Pocket is intended as a protective device for external central lumen catheters, and to provide comfort for the dialysis patient between dialysis treatments. Exterior catheter immobilization provided by the CathGuard™ may reduce discomfort and irritation due to pulling at the central line site. CathGuard™ protects the catheter and supports the catheter in its functionality as a medical product.

Warnings and Precautions:

CathGuard™ is a disposable device and regular replacement at each dialysis treatment is recommended. CathGuard™ is not intended for use as a central line site *dressing* and provides no protection to the site itself. CathGuard™ is non-sterile. Follow the instructions provided by your medical provider for central line dressing care. CathGuard™ is water-resistant but not water-proof.


Instructions for use (see figure below):

1. Carefully place the CathGuard™ pocket over the exterior lumens of the central line catheter.
2. Slide the catheter lumens and caps into the CathGuard™. Avoid pulling on the catheter site.
3. With the catheter lumens in place inside the CathGuard™, remove the two adhesive strip covers on the back of the CathGuard™ and secure it to the patient's skin. Use of a skin prep will improve CathGuard™ adherence and may increase patient comfort by reducing skin irritation.




STEP 1: PLACE ACCESS TUBES INTO CATHGUARD™ POCKET.

STEP 2: REMOVE ADHESIVE STRIP COVERS FROM CATHGUARD™ POCKET AND ATTACH TO SKIN.

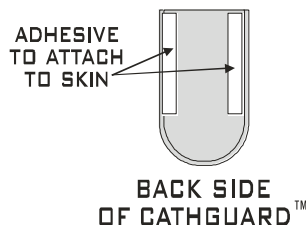

NOTE: REPLACE CATHGUARD™ POCKET WITH EVERY TREATMENT.

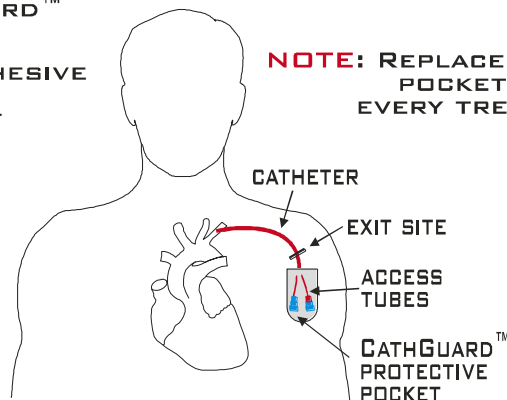

RPC • Tucson, AZ • Minneapolis, MN • USA • (520) 888-5551 • orders@rpc-rabrenco.com

Product Quality at Discount PricingSM