



REF BP00-xxxx

Accu-Read™ Reusable Blood Pressure Cuffs



Instructions for Use

To ensure maximum life/performance of Replacement BP Cuff parts:

- Do not allow cuffs to be inflated when not attached to patient.
- Use correct cuff size based on arm/limb circumference.
- Store safely after each use, keeping cuff off floor.
- Periodically remove lint/fibers from Velcro® portion of cuff.
- Periodically clean external nylon areas of cuff.
- To disinfect RPC cuffs, use 500ppm chlorine solution. It is recommended to use RPC’s K100-0112 Chlorine Potency Test Strip to test the solution to ensure it contains a minimum of 500ppm chlorine.

Not manufactured using natural latex rubber

Part #	Product Description	Packaging
BP00-0105	Small, Adult, 1 Tube, Blood Pressure Cuff (17-26 cm)	5/ca
BP00-0120	Small, Adult, 1 Tube, Blood Pressure Cuff (17-26 cm)	20/ca
BP00-1105	Regular, Adult, 1 Tube, Blood Pressure Cuff (25-35 cm)	5/ca
BP00-1120	Regular, Adult, 1 Tube, Blood Pressure Cuff (25-35 cm)	20/ca
BP00-2105	Large, Adult, 1 Tube, Blood Pressure Cuff (35-45 cm)	5/ca
BP00-2120	Large, Adult, 1 Tube, Blood Pressure Cuff (35-45 cm)	20/ca
BP00-2205	Large, Adult, 2 Tube, Blood Pressure Cuff (35-45 cm)	5/ca
BP00-3105	Thigh, Adult, 1 Tube, Blood Pressure Cuff (45-62 cm)	5/ca