

REF STxxxxFL/STxxxxFLC

SoftCheck® Single Patient – Multiple Use Blood Pressure Cuffs



Instructions for Use

This device complies with American Heart Association recommendations for cuff width to arm circumference ratios if used according to labeling. This product is intended for Single Patient – Multiple Use applications. Accuracy may be affected if used beyond the index line/ range marking parameters.

Read blood pressure monitor’s instruction prior to use.

Select the correct blood pressure cuff based upon limb circumference.

Apply the cuff to the selected limb area.

Properly align the cuff artery markings and snugly secure/ fasten the cuff.

If the cuff markings are out of index range, choose a larger or smaller cuff as required.

Not manufactured using natural latex rubber

Part #	Product Description	Packaging
ST0814FL	BP Cuff, Infant, 8-14 cm, 1 Tube	5/ca
ST0814FLC	BP Cuff, Infant, 8-14 cm, 1 Tube	20/ca
ST1320FL	BP Cuff, Child, 13-20 cm, 1 Tube	5/ca
ST1320FLC	BP Cuff, Child, 13-20 cm, 1 Tube	20/ca
ST1826FL	BP Cuff, Small Adult, 18-26 cm, 1 Tube	5/ca
ST1826FLC	BP Cuff, Small Adult, 18-26 cm, 1 Tube	20/ca
ST2635FL	BP Cuff, Regular Adult, 26-35 cm, 1 Tube	5/ca
ST2635FLC	BP Cuff, Regular Adult, 26-35 cm, 1 Tube	20/ca
ST2938FL	BP Cuff, Long Adult, 29-38 cm, 1 Tube	5/ca
ST2938FLC	BP Cuff, Long Adult, 29-38 cm, 1 Tube	20/ca
ST3242FL	BP Cuff, Large Adult, 32-42 cm, 1 Tube	5/ca
ST3242FLC	BP Cuff, Large Adult, 32-42 cm, 1 Tube	20/ca
ST3544FL	BP Cuff, Large Adult Long, 35-44 cm, 1 Tube	5/ca
ST3544FLC	BP Cuff, Large Adult Long, 35-44 cm, 1 Tube	20/ca
ST4250FL	BP Cuff, Thigh, 42-50 cm, 1 Tube	5/ca
ST4250FLC	BP Cuff, Thigh, 42-50 cm, 1 Tube	20/ca