



REF K100-0104
E-Z Chek® 0 - 14 pH Test Strips




Instructions for Use

Indications for Use

K100-0104 E-Z Chek® 0 - 14 pH Test Strips are indicated for use in testing the pH of water used for preparing dialysate. There are no other indications for use.

Test Procedure

1. Dip into the test solution until the color on strip no longer changes.
2. Gently shake off excess liquid.
3. Compare test pad colors with color scale on bottle.

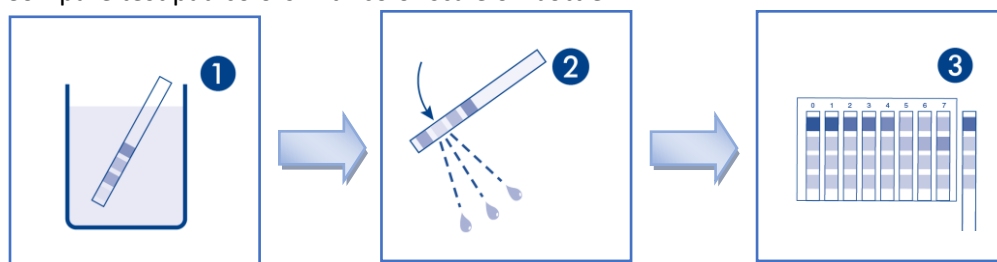

Warnings and Cautions

- Match the strip as closely as possible to one of the color scales on the bottle. Read the pH listed above the matched colors. Colors that are between two scales indicate a pH that is between the two values for those scales.
- Do not touch the indicator pads on the test strip.
- Remove only as many test strips as are required. Close the container immediately after removing a strip.
- Keep all unused test strips in the original bottle.
- Do not allow the test strip to come in contact with liquids or with work surfaces that may contaminate the strip.
- Expiration date and lot number are printed on the container label.
- Verify that the test strips are within their expiration date before use, and do not use them after the expiration date.
- Do not leave test strips in areas exposed to vapors of any type.
- When used as a medical device, Federal Law restricts this device to sale by or on order of a physician.

Interferences

No interferences are known.

Chemical Properties of the Test

OSHA classifies test strips as an article not requiring a Safety Data Sheet (SDS).

Quality Control

RPC performs an independent quality control test on each lot number of test strips (Certi-Chek™ Field Verification Program). RPC includes a certificate of conformance with each shipment per lot and specifies the IFU revision applicable. Certificates of conformance are also available upon request and/or can be downloaded at www.rpc-rabrenco.com.



REF K100-0104
E-Z Chek® 0 - 14 pH Test Strips



Instructions for Use

Storage and Handling

• **Symbol Legend:**

Temperature Limit
39°F (4°C)  86°F (30°C)

